


Macroplastique® Scope Compatibility Chart

The Uroplasty Rigid Endoscopic Needle (part number MRN-518*) is compatible with scopes with a minimum of a 6 Fr inner sheath/working channel and a less than 15° angulation. A 7 Fr inner sheath/working channel and 12° or 30° lens is preferred. Individual experience may differ; Uroplasty, Inc. makes no guarantee of compatibility. Additional scopes may also be compatible.

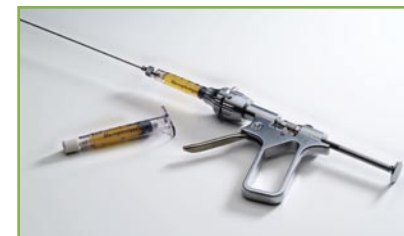

	Part Number	Scope Specifications	Outer Sheath	Inner Sheath/ Working Channel	Notes
Gyrus ACMI	M3-12A	Operative Telescope 12° lens	EVUS-22-ST, 22 Fr	EAWE - Working Element	Requires EVUS-22-OB Urethrotome Visual Oburator
	MRO-2004	Nephroscope	17 Fr	12 Fr	Angled eye-piece
Olympus	A22002A	OES Pro 30° lens, 4mm	Option A) WA22810A, 22.5 Fr Option B) A22026A, 26 Fr	Option A1) WA22820A, 7 Fr Option B1) A42091A, 8 Fr Option B2) A22065A, 7 Fr	
Karl Storz	27005 Series	Resectoscope 0°, 12° or 30° lens, 4mm	27042LI Working Element, 17 Fr	7.2+ Fr	Needs 24 - 26 Fr sheath
	27035BA	Universal Cysto-Urethroscope 30° lens	No sheath needed All-In-One Scope, 17 Fr	7 Fr	MRN-518 fits but may be too tight for easy use
	27035BB	Universal Cysto-Urethroscope 30° lens	No sheath needed All-In-One Scope, 20 Fr	10 Fr	
	27294AA	Nephroscope with angled eye-piece, 6° lens	All-In-One Scope, 17 Fr	10.8 Fr	Hollow Obturator 27294BO should be used
	27295AA	Nephroscope with parallel eye-piece, 6° lens	All-In-One Scope, 17 Fr	10.8 Fr	Straight scope, Hollow Obturator 27294BO should be used

* The MRN-518 Uroplasty Rigid Endoscopic Needle has a 16 ga. shaft with an 18 ga. tip. This is the only needle recommended for use with Macroplastique.

Macroplastique® Scope Compatibility Chart *(cont.)*

	Part Number	Scope Specifications	Outer Sheath	Inner Sheath/ Working Channel	Notes
Wolf	8626.431	5° lens	All-In-One Scope	8625.315 bridge, 7 Fr	Fits on O'Donnell's Ped Cystoscope
	8626.531	E-line Peds Cystoscope 12° lens	All-In-One Scope	8 Fr	
	8632.411	20° lens, 12 Fr	8632.111, 24 Fr		
	8642.403	Compact Urethroscope	All-In-One Scope, 14 Fr		
	8645.413	Atraumatic Urethroscope 25° lens	All-In-One Scope, 21 Fr	7 Fr	
	8645.60	Compact Urethrocystoscope 25° lens	All-In-One Scope, 21 Fr	7 Fr	
	8654.422	30° lens, 21 Fr	8632.024, 21 Fr	8632.025, 21 Fr	8632.124, 21 Fr Obturator 8632.724, 21 Fr Visual Obturator
	8654.431	12° lens, 21 Fr	8632.024, 21 Fr	8632.025, 21 Fr	8632.124, 21 Fr Obturator 8632.724, 21 Fr Visual Obturator



Uroplasty, Inc.

5420 Feltl Road
 Minnetonka, MN 55343 USA
 Tel: 866 258 2182, Fax: 866 255 4522
 e-mail: info.usa@uroplasty.com

www.uroplasty.com

© 2007 Uroplasty. All rights reserved.
 0550021B 11/07

Urgent and Uroplasty are registered trademarks of Uroplasty. The other brands listed are registered trademarks of their respective owners and not the registered trademarks of Uroplasty.

Catalog Number	Product Description
MPQ-2.5	Macroplastique Implants <ul style="list-style-type: none"> • One 2.5 ml unit (2 syringes used per procedure)
Uroplasty Bulking Agent Accessories	
AD-US	Reusable Administration Device <ul style="list-style-type: none"> • Includes syringe adapter
MRN-518	Uroplasty Rigid Endoscopic Needle <ul style="list-style-type: none"> • 5 Fr. Shaft x 15" (380 mm) with 18 Gauge Tip x 0.4" (10 mm)