Male Stress Urinary  
Incontinence and  
Macroplastique®

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 **Uroplasty**

*Dedicated to Improving  
Your Quality of Life*

# Male Stress Urinary Incontinence & Macroplastique

## *The Facts*

Macroplastique bulking agent treatment is designed to treat your stress urinary incontinence and restore your Quality of Life.

With longer life expectancy and better health care facilities, more men are being diagnosed with BPH (Benign Prostatic Hyperplasia) a non-cancerous enlargement of the prostate, or prostate cancer. Curative treatment is offered for BPH and prostate cancer with surgery, but urinary incontinence can be one of the complications. Your doctor has recommended a minimally invasive procedure using a cystoscope to place Macroplastique into the tissue near your bladder as the most suitable treatment for your urinary incontinence.

This minimally invasive Macroplastique procedure is advised for male patients when other non-surgical treatments or medications have failed or were ineffective in treating urinary incontinence as a result of their prostate surgery.

Macroplastique has a clinically documented success rate of approximately 60-65% in this treatment for males, and has demonstrated long-term positive results in comparison to other bulking agents. International Scientific Publications and Abstracts are available upon request.

## *How Macroplastique Successfully Treats Your Stress Urinary Incontinence*

Macroplastique is a soft-tissue bulking agent that adds volume to the tissue surrounding the urethra. This extra volume around the urethra may increase the sphincter muscle closure pressure so you will no longer leak urine during normal everyday activities. Macroplastique, a safe and effective alternative to surgery, has been used for over a decade to successfully treat urinary incontinence for a large number of men worldwide.

### *The Procedure*

The Macroplastique treatment is performed under general anaesthesia as a daycase procedure, and is usually accomplished in less than 30 minutes. This treatment is carried out using a cystoscope to place Macroplastique into the sphincter muscle tissue surrounding the urethra near the bladder. Macroplastique adds bulk to the surrounding tissue resulting in sphincter closure and improved bladder control.

After the procedure, most men can go home the same day, depending on the advice of their doctor, and are able to resume normal activities or return to work shortly thereafter. In some countries an overnight or short hospital stay is normal. You should experience immediate results without complications. If you experience difficulty in passing urine within the first 24 hours, this usually resolves rapidly. You may experience some discomfort or notice a little blood when you pass urine for up to 48 hours after the procedure.

Before consenting to the Macroplastique treatment, it is important to discuss all aspects with your doctor, including possible side effects and care after your procedure. It is essential to follow all instructions the doctor gives you before and after this treatment.

Health and scientific authorities around the world have rigorously evaluated the biocompatibility and safety of Macroplastique. Macroplastique is a safe and beneficial treatment option that will restore independence and Quality to your Life.

## Glossary

### Anaesthesia

Temporary loss of feeling or consciousness caused by administration of a drug.

### Biocompatibility

A material with good biocompatibility is compatible with the body, and does not cause injury or toxic effect to the body.

### Bladder

A muscular organ in the lower abdomen where urine is stored.

### Benign Prostatic Hyperplasia (BPH)

A non-cancerous enlargement of the prostate gland that occurs naturally due to aging.

### Cystoscope

A tube-like optical viewing device inserted through the urethra to examine the bladder.

### Prostate

An organ located at the base of the bladder and surrounds the urethra in males that secretes hormones.

### Sphincter

A group of circular muscles that open or close near the bladder neck or within the urethra.

### Urethra

The tube that carries urine from the bladder out of the body.

## Urethral

Pertaining to the urethra.

## Urinary Incontinence

An involuntary loss of urine.

## Stress Urinary Incontinence (SUI)

An involuntary loss of urine during everyday activities such as laughing, coughing, sneezing, walking, bending or exercising.

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Macroplastique® is a registered trademark of Uroplasty, Inc. U.S. Patent Numbers 5,258,028; 5,336,263 and 5,571,182. United Kingdom Patent Number 2,227,176 and 636014. German Patent Numbers DE 3941 023 C2 and DE 693 18 835 T2. The Netherlands Patent Number 193 399. Canadian Patent Number 2133756. Japanese Patent Numbers 2694372 and 3003724. Additional Patents Pending.
© Copyright 2003 Uroplasty BV. All rights reserved.
**Uroplasty Ltd**